

Intended use:

The infrared forehead thermometer is intended for the intermittent measurement of human body temperature from the skin surface of forehead. The device can be reused by people of all ages for home use and clinical use.

Warning:

1. There is no gender or age limitation for using infrared forehead thermometer.
2. Do not touch the temperature probe with hands .
3. Use of this Forehead thermometer is not intended as a substitute for consultation with your physician.
4. Do not allow children to take their temperatures unsupervised, some parts are small enough to be swallowed.
5. Never immerse this device in water or other liquids.
6. Do not modify this equipment without authorization of manufacturer.
7. Do not expose the thermometer to temperature extremes (below -25°C / -13°F or over 55°C / 131°F) nor excessive humidity ($>95\%RH$).
8. Keep the battery away from children.
9. Remove battery from the device when not in operation for a long time.
10. Do not put the thermometer in direct sunlight or with cotton wool, otherwise the accuracy will be affected.
11. Portable and mobile RF communications can affect the device. The device needs special precaution regarding EMC according to the EMC information provided in the accompany documents.
12. ME equipment should not be cleaned and wiped while in use.
13. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
14. The probe of the ME equipment shall not be serviced or maintained while in use with a patient.